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MEDICAL MASK

EN 14683:2019

Product: Medical mask
Model No.: KN95 mask

Product performance:

1. The use of ultra-fine fiber electrostatic melt-blown cloth composite ES hot air cotton, PP spun bonded non-woven fabric, forming a four-filter layer, more effective filtration of harmful substances.
2. The 3D shape is designed according to the engineering of human faces to ensure the tightness and increased the breathing volume of the mask. This greatly improves the air permeability and make the wearing and breathing more comfortable.

Instruction for use:

1. Open the packaging and place the mask over your nose and mouth.
2. Pull the side straps over your left and right ears.
3. Adjust the nose clip around the bridge of your nose and press it down to fit the shape of your nose.

Storage condition: climatic condition below the temperature $\leq 80^{\circ}\text{C}$. Store the mask in a non-corrosive, ventilated environment.

Usage range:
Suitable for protecting from dust, PM2.5 haze particles, influenza bacteria, and other harmful particles. Complies with the requirements of the Medical devices Directive 93/42/EEC (MDD) and complies with the following European Standards: EN 14683:2019

Manufacturer: PUTIAN JINLILAI CLOTHING & ACCESSORIES CO., LTD.
Address: FENGSHAN VILLAGE, HUANGSHI INDUSTRIAL PARK, LICHENG DISTRICT, PUTIAN CITY

Lot Code: 202000103
Date of production: 2020/03/23
Date of inspection: 2020/03/23
Shelf life: 2 years
Package: 2 Masks per pack
Chinese standard: GB2626-2006
Made in China

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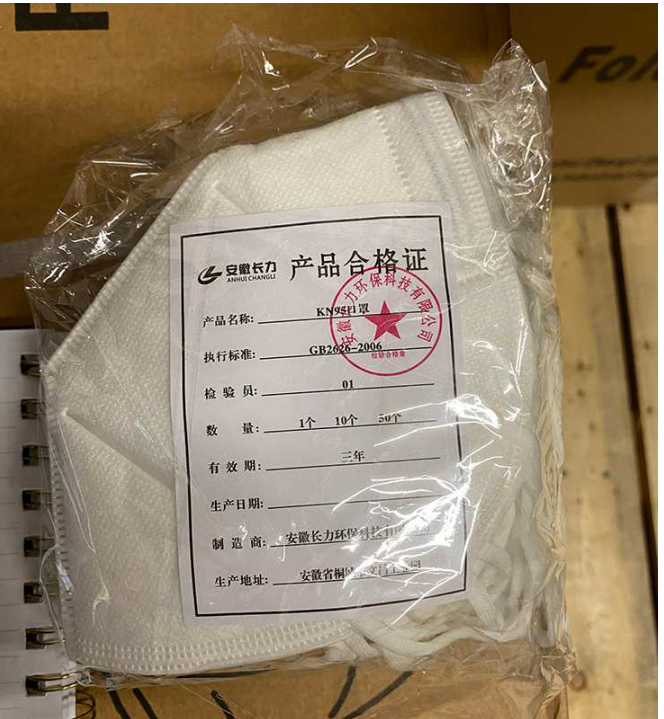
1. Flatten and open the mask
2. Face the face mask without nose clip, so that nose clip above the mask
3. Pull the ear band behind your ear
4. In the middle of the nose clip, press in according to the shape of the bridge of the nose strip from the middle to the two sides until it is fully pressed into the shape of the bridge of the nose. Holding the nose clip with only one hand may affect the tightness of the mask
5. Before entering the work area, the mask must be checked for tightness to the face

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Production date: marked on the certificate of quality

Product material: oil proof non-woven fabric layer melt-blowing fabric, melt-blowing fabric, filter sponge, skin friendly non-woven fabric

Product material: oil proof non-woven fabric layer melt-blowing fabric, melt-blowing fabric, filter sponge, skin friendly non-woven fabric

Manufacturer: Anhui Changli Environmental Protection Technology Co., Ltd

Production address: No.8, Wenchang Industrial Park, Tongcheng City, Anhui Province, China

Notes: This product should not be sold as a surgical mask, as the middle layer of the product is made from melt-blowing fabric which is unable to filter some non-oil particles such as bacteria and dust thoroughly.

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